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# CuroCell® Nova & CuroCell® 2

The CuroCell® Nova and CuroCell® 2 mattresses are suitable for many and are very easy to handle and operate. The CuroCell® Nova have a built-in heel function. They are used preventatively and in the treatment of pressure ulcers up to and including category 2<sup>(1)</sup>.

They are delivered with a durable, soft, elastic hygienic cover that is vapour permeable<sup>(2)</sup>, which is easy to clean and have a very well documented hygienic standard<sup>(3)</sup>.

(1) European Pressure Ulcer Advisory Panel, Quick reference guide 2009.

(2) SS-EN ISO 15496:2004, DIN 53.122.

(3) ISO 16603 "resistance of penetration by blood and body fluids", ISO 16604 "resistance of penetration by blood-borne pathogens", EN 14126, SS 876 00 19 "Health care textiles - Bacteria penetration - Wet".

## Evaluation of the effectiveness of the system

(Care of Sweden, 30/09/09)

Study aimed to describe the function and performance of alternating pressure mattresses feature. This was conducted with test subjects in a weight range between 57 kg to 135 kg. Below are the pressure relief periods presented, ie percentage of a cycle period where the pressure is below threshold values.

### Pressure relief period


59 % below 30 mmHg

43 % below 20 mmHg

28 % below 10 mmHg

### Performance example

The tables show a cycle period at different loads.

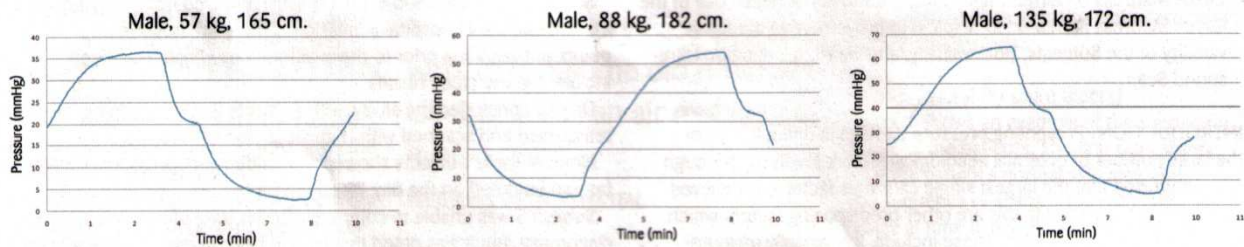## Case Study

### History

Female, 65 years old, with angina pectoris, myocardial infarction and dementia. Pressure ulcer, category 2 on both lower legs, R/T the use of devices for heel protection. Point according to the Modified Norton scale is 14, BMI is 22. She has a normal diet but has eaten and drunk less recently. Urinary incontinence, uses diapers.

28/08/07

### Actions

She was ordained an alternating pressure relief mattress, CuroCell Nova®, all other aids were removed. The patient received daily supplements of nutrient drinks. After five weeks, the damage was completely healed.

5 weeks later